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POWDERCRUSH.se

Instructions for use

Later during 2006, instructions comes in this languages: Norwegian, Danish, Finnish.

Before You Start

Your **POWDERCRUSH™** pill crusher, comes complete with batteries installed, and is supplied with a plug-in charger. Ensure that you have received your crusher complete, and in good condition. If there are any deficiencies or damage, contact your dealer.

To Operate

Before operating the **POWDERCRUSH™**, locate the switch at the underside of the machine, and slide it to the "on" position. With the power switch on, plug the charger into the appropriate wall receptacle, and insert the charge plug into the power jack. The jack is located at the end of the crusher, below the "start" button. The LED at the top of the **POWDERCRUSH™** will glow red, when charging is in progress. Allow approx 3 hours to bring the batteries to a full charge.

The **POWDERCRUSH™** should be operated on a flat surface. Place pills to be crushed, in a Nordic Medpartner supplied pill crusher pouch, one pill at a time, ensuring that they are lying flat against the pouch. For best results, allow a minimum of 1 inch between the tablets and the pouch opening. Fold over the top of the pouch to prevent residue from "puffing" out. A double fold is recommended. Use only the recommended Nordic Medpartner supplied pill crusher pouches for **POWDERCRUSH™**.

Insert the pill crusher pouch filled with pills in the slot at the top of the machine with one hand, while holding down the start button with the other hand. **KEEP FINGERS OUT OF SLOT.** When the **POWDERCRUSH™** is running, the LED shows green. Typically, 4-5 seconds will produce the desired results, when 2 or 3 pills are in the pill crusher pouch. For a finer crush, operate for a longer period. Do not operate the **POWDERCRUSH™** when not crushing pills.

POWDERCRUSH™ performance will be affected when the battery charge is low. To recharge, return to the battery charger. It is recommended that you recharge your **POWDERCRUSH™** after every medication pass.

POWDERCRUSH™ should be cleaned with a non-residue cleaner such as alcohol.

To Replace Battery Packs

Ni-Cad batteries will degrade in time. This will be evident when the battery fails to hold a full charge. The **POWDERCRUSH™** uses 2, 12-volt NiCad power packs that are supplied by Nordic Medpartner. They can be accessed and replaced by removing the 6, Philips head, machine screws from the machine base, on the underside of the crusher. Carefully unplug the terminals of the pack by pushing down on the miniature plug "lock" and pulling out. To install, simply push the 2-prong battery plug into the mating connector. Ensure the cables are contained, before replacing the base plate. Re-install the six screws. Do not over-tighten.

Specifications

Height: 9cm
Width: 8cm.
Length: 28cm
Weight: 2,7kg
Battery: 2, 12 volt NiCad, 800 mAh
Charger: 220 volt / 16volt/300ma output

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Copyright 2005 Nordic Medpartner AB